


Recommended Application Instructions for RFID Tags

Apply Allflex Electronic ID Tags using the appropriate tag applicator. When using the Universal Total Tagger (Red), use the Red Blunt Pin and remove the Black Insert from the base of the jaws.

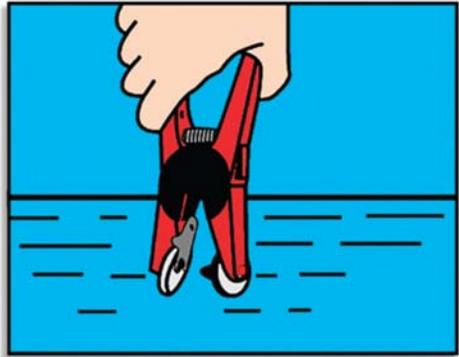


1. To load, depress spring clip and insert the female RFID tag. Ensure that the raised portion of the tag, which encloses the transponder chip, is placed in the open portion of the jaws.

2. Slip the male tag completely onto the blunt applicator pin. Squeeze the jaws together lightly to ensure the male shaft is in line with the female.

3. Dip the jaws of the applicator holding the tag into an antiseptic or disinfectant solution.

Recommended RFID Tag Placement for Cattle

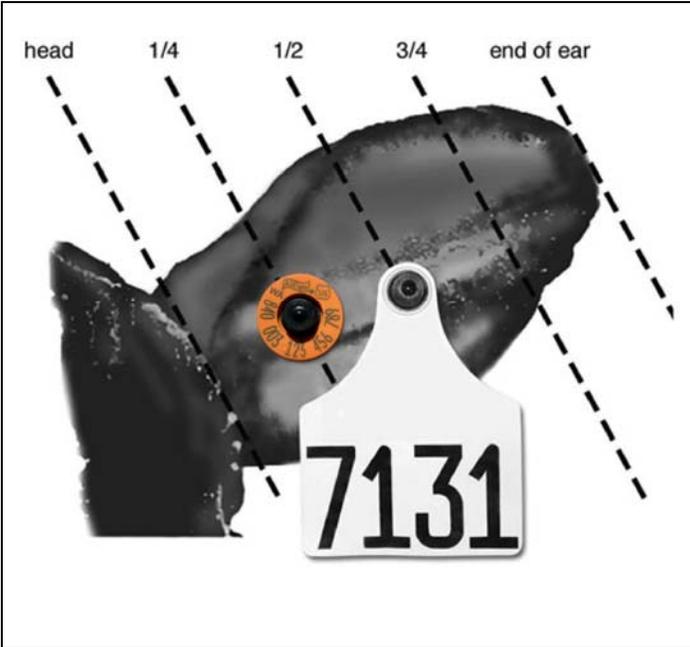

Application site must be free of foreign debris prior to placement of tag on the animal. Review application instruction prior to tagging.

1. The RFID tag is placed in **either** ear, within the first quarter between the two cartilage ribs. Be careful not to cover the Brucellosis tattoo when applying the tag to the right ear. An option when vaccinating is to place the tattoo below the bottom rib in the ear.
2. The female portion of the tag should be on the inside of the ear with RFID tag application. Note that this is a thicker part of the ear. Application may be more difficult than when applying a visual tag.

IMPORTANT: Caution, "Free Air Space" is critical for proper healing and retention. Inspect placement after tagging to ensure there is sufficient space between ear and RFID tag.

Record the full 15-digit number onto the Federal Vaccination record or other official forms where Official ID is required.

Special thanks to Allflex for sharing the use of their tag application content.